

## DR. NASSEH ENDODONTIC MICROSURGERY SET



A focused,  
minimalistic  
instrument set  
supporting key  
microsurgical  
procedures:

- Apicoectomy
  - Retrofilling
  - Surgical root perforation repair
- ...and more

Rooted in the Surgical Lid-Technique  
developed by Dr. Allen Ali Nasseh.  
A modern approach that enhances  
efficiency, safety and precision  
in every endodontic surgery — from  
apicoectomy to root and perforation repair.



**SIMPLICITY. EFFICIENCY. SAFETY.**

SET NO: 8314

9218

Mundspiegelgriff, einendig, TRINOVO PEEK  
Mouth mirror handle, single-ended, TRINOVO PEEK  
Manche miroirs, simple extrémité, TRINOVO PEEK  
Mango para espejo bucal, un extremo, TRINOVO PEEK



4516

Mundspiegel Rhodium Gr. 5 (Ø24mm)  
Mouth Mirrors Rhodium No. 5 (Ø24mm)  
Miroir Rhodium plan n° 5 (Ø24mm)  
Espejos de boca, plano, Rhodium n° 5, (Ø24mm)



9218-DE

Mundspiegelgriff doppelendig, TRINOVO PEEK  
Mouth mirror handle double end, TRINOVO PEEK  
Manche miroirs double extrémité, TRINOVO PEEK  
Mango para espejo bucal doble extremo, TRINOVO PEEK



6412-CC

Mikro-Mundspiegel, Ø3 mm  
Micro mouth mirror, Ø3 mm  
Micro miroir, Ø3 mm  
Micro espejo bucal, Ø3 mm



6414-CC

Mikro-Mundspiegel, oval 3x6 mm  
Micro mouth mirror, oval 3x6 mm  
Micro miroir, 3x6 mm  
Micro espejo bucal, oval 3x6 mm



7693

Skalpellingengerhalter TRINOVO, gerade  
Scalpel handle TRINOVO, straight  
Manche bistouri TRINOVO, droit  
Mango para hojas de bisturi TRINOVO, recto



8428

PA-Sonde CP15/  
Zahnsonde DG16  
Perio Probe CP15/  
Explorer DG16  
Sonde paro CP15/  
Sonde DG16  
Sonda parodontal CP15/  
Sonda DG16



9203

Mikro-Stopfer D1, DLC beschichtet  
Micro plugger D-1, DLC coated  
Micro fouloir D1, revêtement DLC  
Compactador micro D1, recubierto con DLC



8461-CC

Instrument zur Reparatur von Perforation, DLC beschichtet  
Perforation repair instrument, DLC coated  
Instrument pour la réparation de perforations, revêtement DLC  
Instrumento para la reparación de perforaciones, recubierto con DLC



9557

Raspatorium B1, TRINOVO PEEK  
Periosteal elevator B1, TRINOVO PEEK  
Décolleur B1, TRINOVO PEEK  
Periostótomo B1, TRINOVO PEEK



8445

Raspatorium 2/4, DLC beschichtet  
Periosteal elevator 2/4, DLC coated  
Décolleur 2/4, revêtement DLC  
Periostótomo 2/4, recubierto con DLC



8449

Raspatorium MOLT 9, DLC beschichtet  
Periosteal elevator MOLT 9, DLC coated  
Décolleur MOLT 9, revêtement DLC  
Periostótomo MOLT 9, recubierto con DLC



9220

Raspatorium PRICHARD, DLC beschichtet  
Periosteal elevator PRICHARD, DLC coated  
Décolleur PRICHARD, revêtement DLC  
Periostótomo PRICHARD, recubierto con DLC



3308

Ringkürette, Ø2 / Ø3 mm, DLC beschichtet  
Ring curette, Ø2 / Ø3 mm, DLC coated  
Curette annulaire, Ø2 / Ø3 mm, revêtement DLC  
Cureta anular, Ø2 / Ø3 mm, recubierto con DLC



8442

Kürette posterior, DLC beschichtet  
Posterior Curette, DLC coated  
Curette postérieure, revêtement DLC  
Cureta posterior, recubierta con DLC



8401

Wangenhalter MINNESOTA  
Cheek retractor MINNESOTA  
Ecarteur de MINNESOTA  
Separador MINNESOTA



8401-N

Wangenhalter MINNESOTA mit 2,0 mm Vorsprung  
Cheek retractor MINNESOTA with 2.0 mm protrusion  
Ecarteur de MINNESOTA avec protrusion de 2.0 mm  
Separador MINNESOTA con protrusión de 2.0 mm



4075

Pinzette PERRY, gerieft, gebogen, 13 cm  
Pliers PERRY, serrated, curved, 13 cm  
Précelle de PERRY, rainurée, courbée, 13 cm  
Pinza PERRY, estriada, curva 13 cm



4078

Pinzette COLLEGE, gerieft, gewinkelt, 15 cm  
Pliers COLLEGE, serrated, angled, 15 cm  
Précelle de COLLEGE, striée, angulée, 15 cm  
Pinza COLLEGE, estriada, angulada, 15 cm



4018

Schere IRIS, gerade, 11,5 cm  
Scissors IRIS, straight 11,5 cm  
Ciseaux IRIS, droit 11,5 cm  
Tijeras IRIS, recta, 11,5 cm



7664

TC-Nadelhalter CASTROVIEJO mit dickem Maul, gerade, 14 cm  
TC-Needle Holder CASTROVIEJO with thick jaws, straight, 14 cm  
TC- Pte-aig. CASTROVIEJO avec mors épais, droit, 14 cm  
TC- Micro porta agujas CASTROVIEJO, 14 cm



7609-BL

optional

Steritray MAXI mit Steg längs, blau, 275 x 180 mm  
Steritray MAXI with longitudinal bar, blue, 275 x 180 mm  
Steritray MAXI avec entourage transversale, bleu, 275 x 180 mm  
Steritray MAXI con traba transversal, azul, 275 x 180 mm



## Clinical Fields of Application:

- 1 Access to surgical site
 3 Retrofilling
- 2 Apicoectomy
 4 Surgical Repair of root defects/perforations

Article	Instrument	Application	1	2	3	4
7609-BL	Steritray MAXI with longitudinal bar,	Holds the 18 instruments composing the kit. Open face and simple access allows quick set up and clean up.	x	x	x	x
9218	Mouth mirror handle PEEK single ended	An ergonomically designed universal mouth mirror handle. The black color effectively reduces light reflection, ensuring clear visualization during surgical procedures .	x	x		x
4516	Mouth mirrors, rhodium, no. 5 (Ø24mm), PU: 12 pieces	The rhodium mouth mirror provides bright, true-color, distortion-free reflection for precise indirect vision.	x	x		x
7693	Scalpel handle TRINOVO straight	A round ergonomic scalpel handle designed for various flap incisions. It allows for enhanced maneuverability, enabling more precise and controlled incisions.	x			
8428	Perio Probe CP15/Explorer DG16	The explorer DG16 is used to sound the bone and detect fenestrations caused by the lesion. The probe CP15 is used to evaluate the depth of the drilling into the osteotomy, in correlation with CBCT analysis.	x	x		x
9557	Periosteal elevator B1	A double end flap elevator used for smaller, detailed flap elevations in either submarginal or intrasulcular incisions.	x			
8445	Periosteal Elevator 2/4 ENDOBLACK	A double end flap elevator used for the detailed and small-scale elevation of flap edges and papilla in submarginal or intrasulcular incisions. Essential for precise flap elevation in delicate surgical areas.	x			
8449	Periosteal elevator MOLT 9, ENDOBLACK	This double end flap elevator is used after a Buser #1 for a broad scale elevation for elevating flap edges and papilla.	x			
8401	Cheek retractor MINNESOTA	Conventional Minnesota retractor used for flap retraction during surgical flap procedure	x			x
8401-N	Retractor MINNESOTA acc. to Dr. Nasseh with 2,0 mm protrusion	Designed for use with a conventional retractor, this instrument securely retracts soft tissue by providing a stable point of support through its integrated protrusion. Its improved grip minimizes slippage and helps reduce hand fatigue during prolonged retraction.	x			x
3308	Ring curette, Ø 2 / Ø 3 mm	Designed to efficiently remove granulation tissue and cysts while scraping the bony crypt clean.	x			
8442	Posterior Curette, ENDOBLACK	Double end curette for cleaning granulation tissue tags around and behind roots during curettage of the lesion, before and after access to root end.	x			x

Article	Instrument	Application	1	2	3	4
9218-DE	Mouth mirror handle, PEEK double ended	This double end ergonomically designed mouth mirror handle offers versatile functionality, accommodating a range of mirror sizes for enhanced flexibility during procedures. The black color effectively reduces light reflection, ensuring optimal, distortion-free visualization.			X	X
6412-CC	Micro mouth mirror, Ø 3 mm, PU: 1 piece	Round micro-mirror for visualization of the apex throughout the apicoectomy, retrograde filling or root repair procedure.			X	X
6414-CC	Micro mouth mirror, oval 3x6 mm, PU: 1 piece	Oval-shaped micro-mirror for visualization of deeper crypts, where the root is far from the cavity area of the bone. Used throughout the apicoectomy, retrograde filling, or root repair procedure.			X	X
9203	Micro plugger D1 ENDOBLACK	Double end plugger designed used for validating apical preparation size and depth, followed by compacting retrofilling material into the preparation.			X	X
8461-CC	Perforation repair instrument	Used to deliver bioceramic putty and insert it into the apical preparation for the surgical Lid Technique			X	X
9220	Periosteal elevator PRICHARD, ENDOBLACK	Used for flap elevation and carrying bone graft materials into the surgical site. The flat end can also be used for flap retraction or protection during suction.	X			
4075	Pliers PERRY	Used to remove granulation tissue, biopsy samples, and resected root ends, as well as to manipulate tissue and approximate flaps before suturing. It offers a firm yet delicate grip with ergonomic comfort and control, helping maintain a clear surgical field during site preparation after apicoectomy or root perforation	X			
7664	TC-Needle Holder CASTROVIEJO with thick jaws, straight	Needle holder used to suture flap into place. Smaller size and precision tip allows comfortable handling and precision in suturing.	X			
4018	Scissors IRIS, straight	Used for cutting sutures or other materials during the procedure.	X			
4078	Pliers COLLEGE	Non-locking cotton pliers used for carrying gauze or pellets to the surgical site or for removing tissue/materials.	X	X	X	X

Place your order here



info@kohler-dentale



Dr. Allen Ali Nasseh

**What Dr. Nasseh thinks about it...**

“With this refined and focused set of instruments, I can achieve optimal surgical results. My goal in developing this kit was to provide colleagues worldwide with reliable tools that make microsurgery more efficient and predictable. I am glad to share my experience and further guidance with you, so that together we can advance clinical outcomes for our patients.”

allen\_ali\_nasseh | kohler\_dental



@AANasseh | @kohlerdental



**Kohler Medizintechnik GmbH**  
 Bodenseedalle 14-16 | 78333 Stockach - Germany  
 Phone: +49 7771 64999-0 | www.kohler-dentale.de | info@kohler-dentale.de